


Alexer[®]

Protective Backing

- High MVTR
- Smooth and comfortable
- Waterproof and bacteria barrier

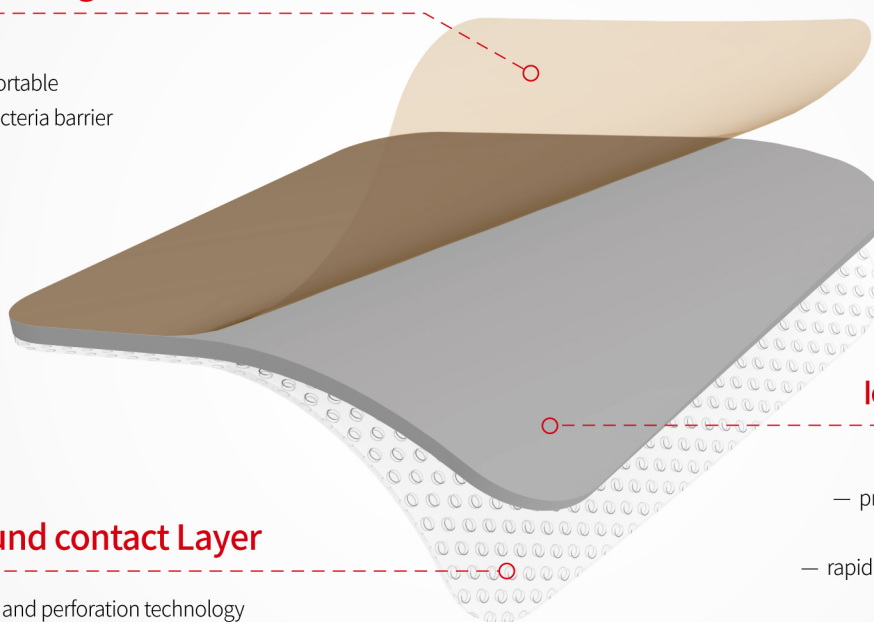

Silicone Wound contact Layer

- Unique silicone oil and perforation technology
- Distributes wound exudate fast and evenly
- Gentle and secure with pain free removal
- Does not strip away epidermal cells

Foam absorbent layer containing Ag

- prevent wound maceration
- provide a moist wound environment
- reduce the risk of infection
- rapid sustained release ionic silver with sustained effect up to 7 days

SILICONE FOAM Ag DRESSING

◇ Without Border

DESCRIPTION

Alexer® silicone foam Ag dressing-without border utilize an innovative three-layer design that improves patient comfort, reduces complications and tissue damage during removal, and creates an ideal healing environment. The silicone adhesive helps the dressing stay in place during bandage wrapping and is an ideal choice for management of moderate to heavily exuding wounds under compression.

How it works?



Alexer® silicone foam Ag dressing-without border absorb exudate and evenly distribute pressure at the wound site through a three-layer system:


Silicone Wound Contact Layer: Unique silicone oil and perforation technology come in direct contact with the wound, allowing for straightforward application, and won't stick to the wound, thus providing nearly painless, atraumatic removal. Once applied, it allows the exudate to quickly migrate to the layers above for fast absorption.

Absorbent Polyurethane Foam: Cushioning protection that provides additional comfort while offering great fluid handling capabilities. Rapid release ionic silver with sustained effect up to 7 days.

Protective Backing: Smooth and comfortable material that is waterproof and a barrier to bacteria while promoting high moisture vapor transmission rate (MVTR). Dressing lays smooth around wound edges.

BENEFITS

- Won't leak or loosen when lifted or repositioned
- Prevents tissue from growing into the dressing, thanks to the perforated texture
- Lessens irritation and pain during removal and helps with reducing potential complications
- Can be worn for up to 7 days
- High breathability (MVTR) supports good moisture balance for wound healing
- Waterproof, bacteria proof, and air-permeable

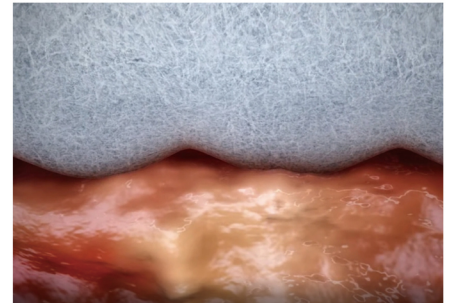

Indications of Alexer® SILICONE FOAM Ag DRESSING without Border

SIZE: 10X10cm,15X15cm
PACKAGE: 5pcs/bx, 40bxes/ctn
Ref.No.: S12012001,S12012002



- ✓ Pressure ulcers
- ✓ Leg and foot ulcers
- ✓ Traumatic wounds
- ✓ First and second-degree burns
- ✓ Surgical wounds
- ✓ Donor sites
- ✓ Skin tears
- ✓ Dry or necrotic wounds, when used in combination with a wound gel
- ✓ Acute and chronic wounds

