





Alexer[®]



SILICONE SCAR GEL

-  **MEDICAL-GRADE** | 100% liquid silicone
-  **DISCREET** | Clear & odorless
-  **SOOTHING** | Relieves itchiness & discomfort
-  **SAFE** | Suitable for pediatric & facial scars



DESCRIPTION

A silicone creamy gel formula creates a barrier to help prevent new scars from forming, while improving the appearance of existing scars. Ideal for superficial and post-procedure scars, as well as cuts, scrapes, and burns.

- ✔ Superficial scars are left visibly smoother and softer.
- ✔ Provides a protective film over the skin.
- ✔ Helps reduce the appearance of scarring.



DIRECTIONS FOR USE

- Should only be used on healed wounds, never on broken or unhealed skin.
- The area must be clean and free of any creams. Skin must be washed every day.
- When first using silicone gel please build up a tolerance by gradually increasing the time worn.

Day 1
2 hours


Day 2
4 hours

Day 3
8 hours

Day 4
overnight

PREVENT AND TREAT HYPERTROPHIC SCARS AND KELOIDS

- 01 Softens and flattens raised scars
- 02 Fades red of dark scars
- 03 Relieves the itching and discomfort of scars

Type	Size	Package	Ref.No.
 Scar Gel	0.18 OZ (5g)	1tube/box	S14011001
	0.53 OZ (15g)	1tube/box	S14011002
	1.05 OZ (30g)	1tube/box	S14011003